



## **Accu-Chek<sup>®</sup> Mobile – at a glance**



### **What is Accu-Chek Mobile?**

Accu-Chek Mobile is the new strip-free blood glucose monitoring system that takes the simplicity and convenience of testing to a whole new level. With 50 tests offered on a continuous tape and an integrated lancing device with 6 lancets in a drum, Accu-Chek Mobile enables testing virtually whenever and wherever needed. With this new technology, people with diabetes do not have to worry about the handling of single test strips or single lancets. Waste handling after a test as well, is no longer an issue. Fifty pre-loaded and automatically coded tests are provided on a continuous tape stored safely in a test cassette. The system combines precise and reliable monitoring functions with easy-to-access information management features and uses the trusted Accu-Chek *Clix*motion lancing technology.

### **Why is it a new way of testing?**

Accu-Chek Mobile works with 50 tests on a continuous tape. After a test, the tape automatically advances to the next test area and the system is ready for a new blood glucose test. The tape is safely stored in the test cassette, which can easily be disposed of after all 50 tests are used. Also, Accu-Chek Mobile has a lancing device attached, the Accu-Chek FastClix Mobile equipped with 6 lancets in a drum, making the handling of single lancets obsolete. Thus Accu-Chek Mobile offers its users all the supplies and features they need for frequent and structured self-monitoring of blood glucose in one single device:

- 50 tests on a continuous tape and 6 lancets in a drum allow for more flexibility and on-the-go blood glucose monitoring. Plus a short test time of 5 seconds makes testing even quicker.
- No waste to handle after each test.
- Only a small blood sample of 0.3µl is needed.
- 500 readings can be stored with time and date and averages over the last 7, 14, and 30 days on all values, before or after meal can be calculated.
- Pre- and post-meal markers can be set as well as 5 individually programmable test reminders.

### **How does the Accu-Chek Mobile system support diabetes self-management?**

Accu-Chek Mobile makes testing particularly easy because it only takes **a few simple steps** to get the blood glucose reading:

- open the tip cover,
- obtain the blood sample,
- apply the sample to the test area
- and - after just 5 seconds - read the result.

To prepare for the next test, change to the next lancet by an easy push of the tab on the lancing device. With full connectivity to the Accu-Chek information management solutions as e.g. the Accu-Chek Smart Pix device reader and Accu-Chek 360° software and a download functionality to Microsoft Excel, Accu-Chek Mobile provides quick access to the relevant data. It therefore offers a comprehensive overview for people with diabetes and their caregivers to make optimal therapy adjustments.

## **Accu-Chek<sup>®</sup> Mobile – A new way of testing.**